

the SAM Junctional Tourniquet (SJT)



the SAM Junctional Tourniquet (SJT)



① 圧迫したい部位にTCDが来るようにSJTを体幹に巻く。創部には止血用資材もしくは滅菌ガーゼを当てておく



② TCDがしっかり当たるよう、バックルを閉じる



③ 茶色の方のバックルからクリック音が聞こえるまで両方から引っ張る。




④ 出血が止まるまで送気球からTCDに空気を入れる

the SAM Junctional Tourniquet (SJT)



the SAM Junctional Tourniquet (SJT)

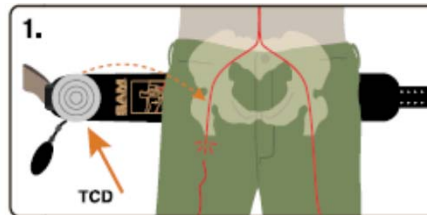

1. Slide the belt underneath the patient, positioning the Target Compression Device (TCD) over the area to be compressed. Use sterile gauze or hemostatic dressing if targeting directly over a wound. For bi-lateral application, use a second TCD.



2. Hold the TCD in place and connect the belt using the buckle.



3. Pull the **BROWN HANDLES** away from each other until the buckle secures. You will hear an audible click. Fasten excess belt in place by pressing it down on the Velcro. You may hear a second click once the belt is secure.



4. Use the hand pump to inflate the TCD until hemorrhage stops. Monitor patient during transport for hemorrhage control and adjust the device if necessary.

TO REMOVE. unbuckle the belt.

http://www.sammedical.com/wp-content/uploads/2013/09/SJT-206-BRO-4_web.pdf

SAMSリング



T-POD

